


FLUSH MOUNT DOOR DETAIL

VPL-3200B SERIES VERTICAL PLATFORM LIFT

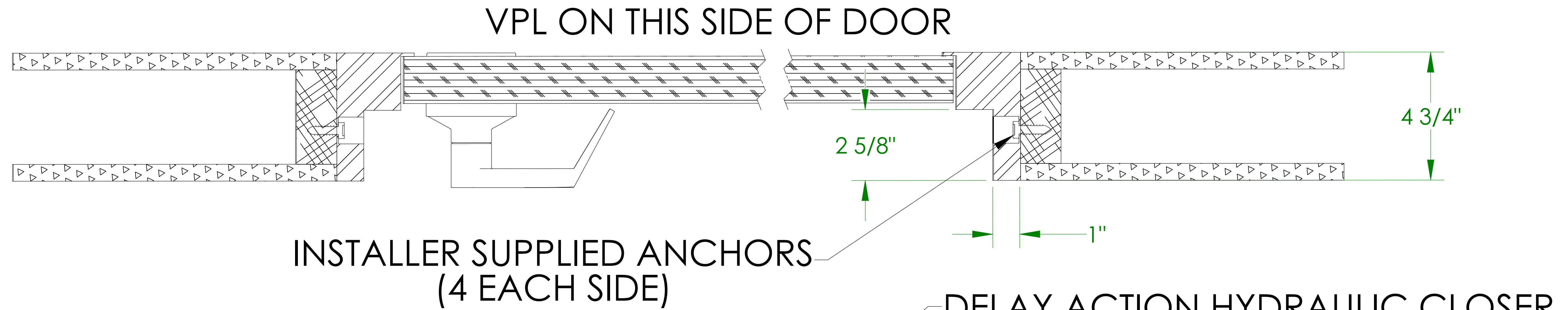


PART NUMBER	DESCRIPTION
VPL-32236	FLUSH MOUNT OAK DOOR (LH SWING)
VPL-32237	FLUSH MOUNT OAK DOOR (RH SWING)
VPL-32238	FLUSH MOUNT STEEL DOOR (LH SWING)
VPL-32239	FLUSH MOUNT STEEL DOOR (RH SWING)


KEYED HANDLE

NOTES:

1. DOOR FRAME IS SHIPPED WITH TACKPLATE SILL WHICH IS REMOVED DURING INSTALLATION (USE A HAMMER TO REMOVE).
2. INTERLOCK IS SHIPPED SEPARATELY TO PROTECT INTERLOCK. INSTRUCTIONS ARE INCLUDED WITH THE INTERLOCK FOR INSTALLATION INTO DOOR FRAME.
3. FINISH:
 - A. OAK DOOR IS PLAIN (NOT FINISHED) AND FRAME IS GRAY PRIMER FINISH.
 - B. STEEL DOOR AND FRAME HAS A GRAY PRIMER FINISH.
4. IF DOOR IS USED WITH A FIXED RAMP, RAISE THE DOOR FRAME UP THE HEIGHT OF THE FIXED RAMP. PLEASE ALLOW FOR THIS IN THE ROUGH OPENING IF NEEDED. YOU WILL HAVE TO FINISH THE BOTTOM OF THE DOOR FRAME AS NEEDED.

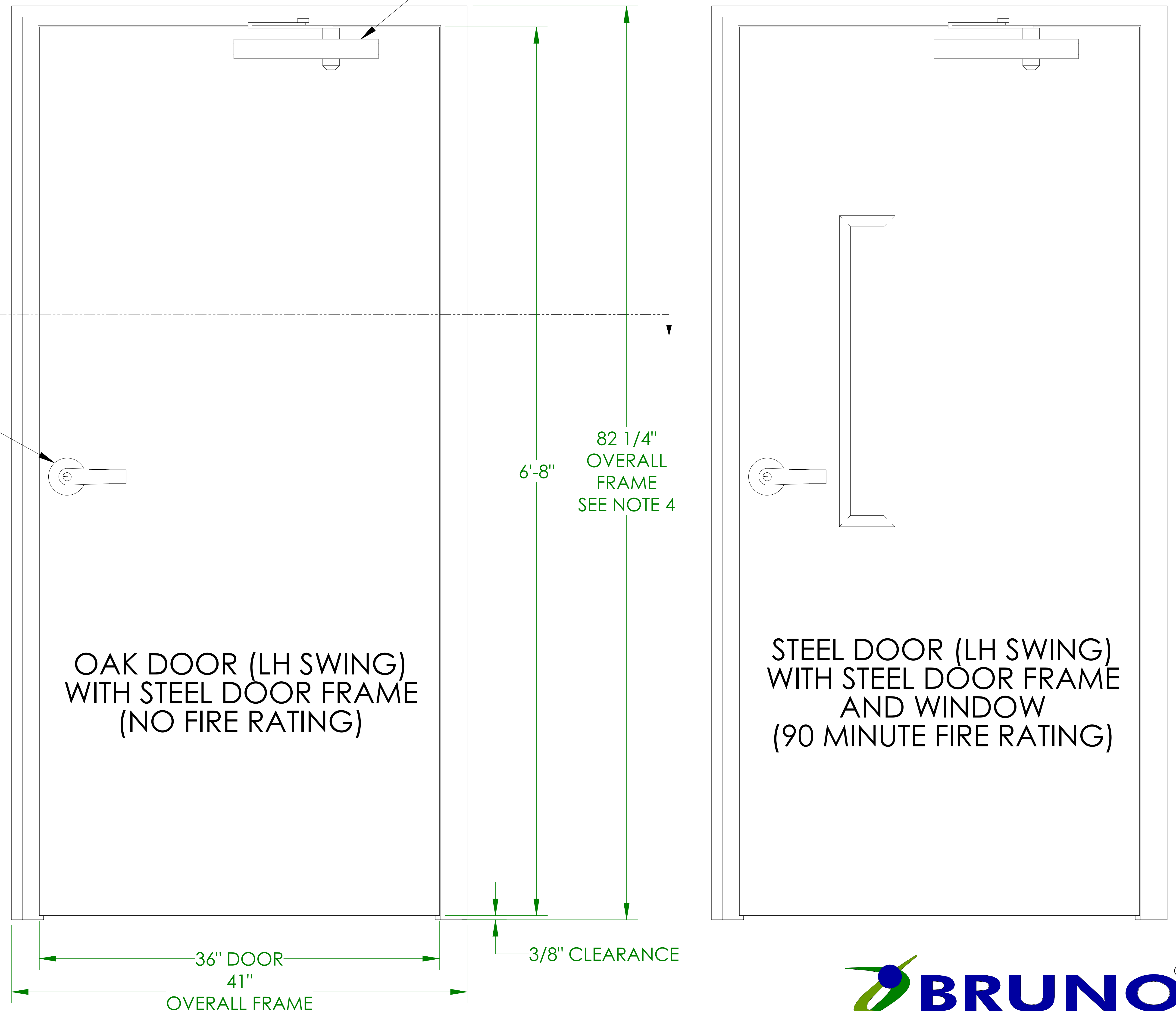


FLUSH MOUNT DOOR ALIGNMENT (STRAIGHT-THRU PLATFORM) VPL-3200B SERIES VERTICAL PLATFORM LIFT

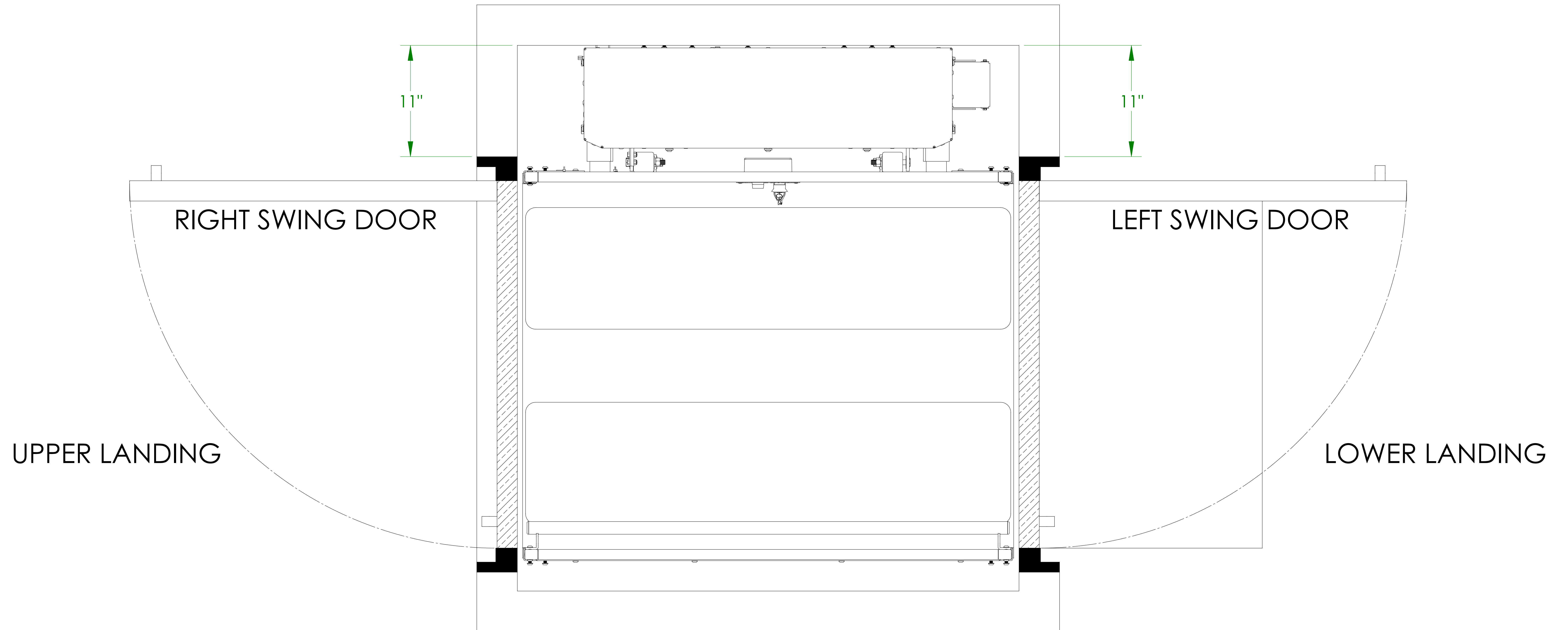

PLAN VIEW
(RIGHT HAND UNIT SHOWN)


ILS-01027
SHEET 2 OF 4
REV. 1 (4894)(11/1/11)(DPG)

FLUSH MOUNT DOOR ALIGNMENT (90° / ADJACENT EXIT PLATFORM) VPL-3200B SERIES VERTICAL PLATFORM LIFT

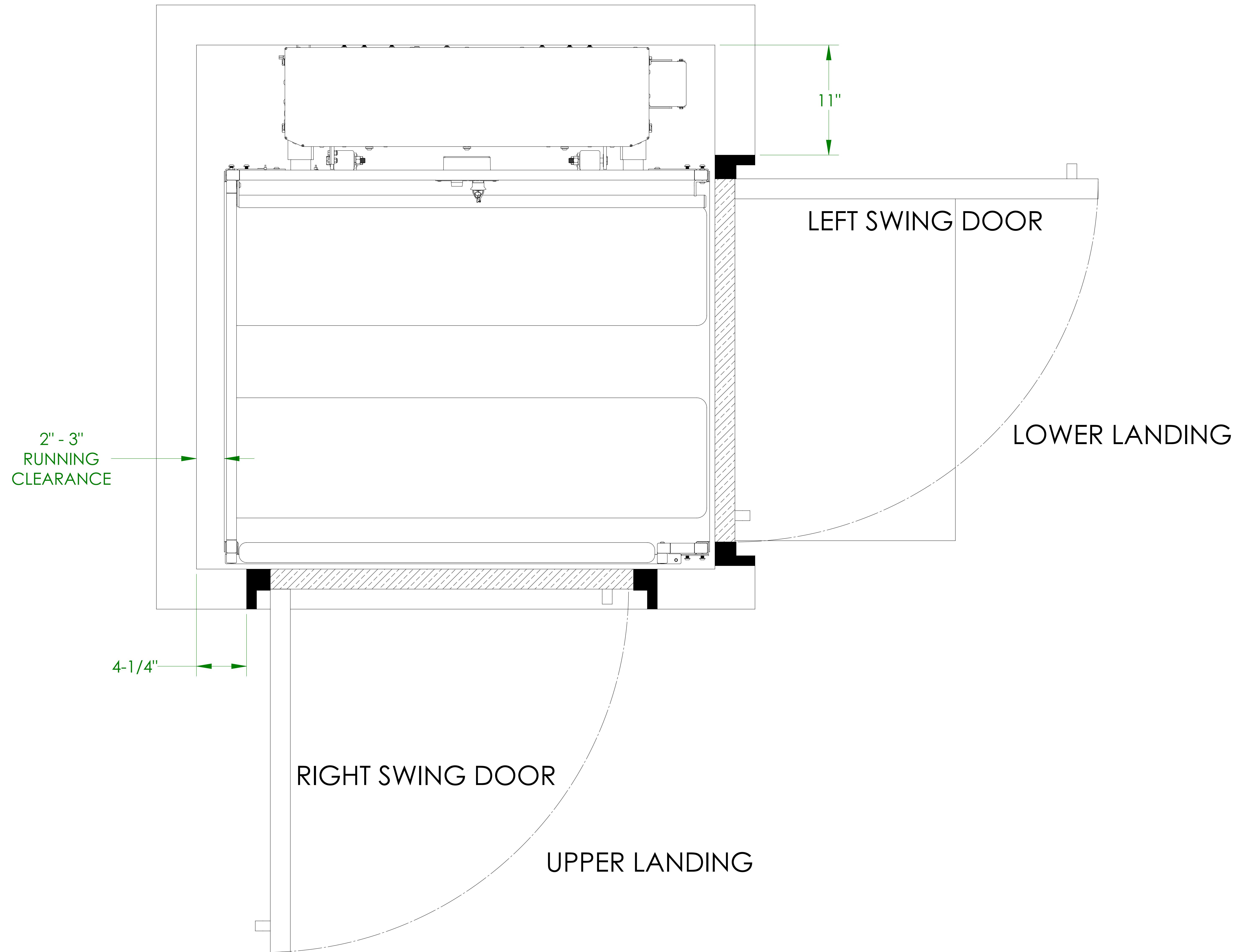

PLAN VIEW
(RIGHT HAND UNIT SHOWN)

FLUSH MOUNT DOOR ALIGNMENT (SAME SIDE PLATFORM) VPL-3200B SERIES VERTICAL PLATFORM LIFT

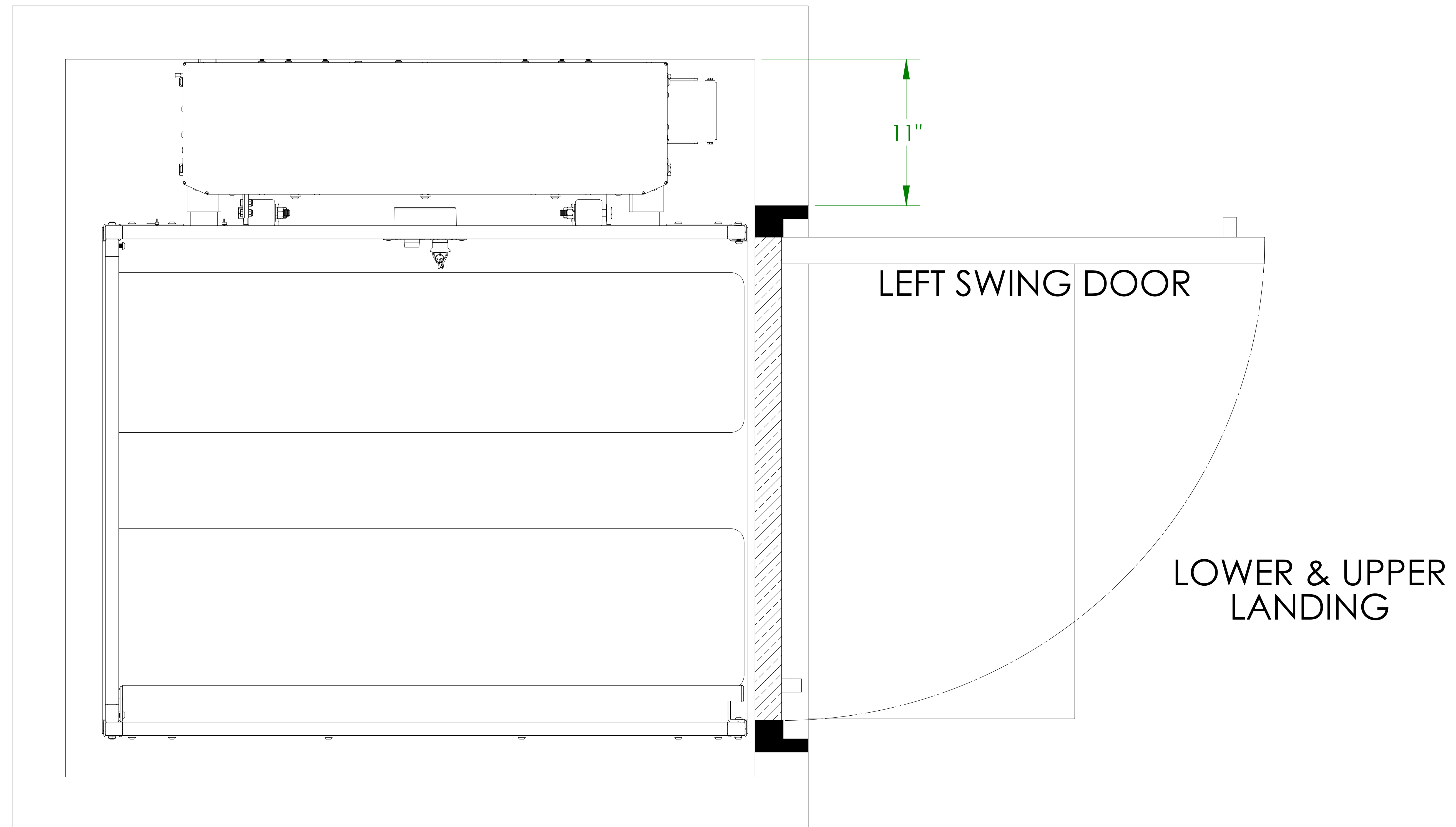

PLAN VIEW
(RIGHT HAND UNIT SHOWN)